

# Dri-Scope Aid® 2

## SD-U Quick Start Guide



1) Open new tubing package(s). Attach time and date sticker directly to the tubing. Install air tubing set to either of the air output ports located on the lower front of the unit.



2) Install the included air output cap on the opposite port if only running one endoscope.



3) If running two endoscopes, install a second tubing set in the alternate air output port.



4) Open new connector kit and install port connector on the tubing set. Gently thread on luer-lock connector.



5) Use + or - buttons to adjust airflow cycle time if desired. Time shown in minutes.



6) Press START button to begin airflow.



7) Upon airflow cycle time completion, the unit will automatically cease operation. Select RESET to restart the timer for the next cycle.



8) Remove luer-lock connector and discard.  
9) Replace tubing set daily.